



How to Use a KN95



To Apply:

1. Wash hands thoroughly before putting on a mask.
2. Place over the face with the outer side facing outwards and the metallic strip on top.
3. Position the elastic bands around both ears.
4. Shape the metallic strip to contour over the nose, so that the mask has a snug fit. Please check for air tightness, and ensure the mask is tightened securely to the face.

To Remove:

1. Hold both ear loops and pull away from the face gently.
2. Avoid touching the outside of mask during removal.
3. Discard the mask appropriately in a trash bin and wash hands thoroughly.

